



TEN ACES MED
STICK WITH US

THE MEDICAL ADHESIVE SPRAY YOU'VE BEEN SEARCHING FOR

At Ten Aces Med, it is our deep belief that everyone deserves to start, live, and end every day filled with confidence.

Our commitment to exceptional customer service, unparalleled product quality, and ongoing innovation ensures the best possible solution for your ongoing medical adhesive needs.



Medical Adhesive Spray
Net Vol. 100 ml (3.38 oz)

THE "STICK WITH US" DIFFERENCE

Made here in America, our silicone based medical adhesive is designed for use with ostomy appliances, wound care, and prosthetics applications.

What sets us apart is our 360° any angle spray, making it easy to apply in hard to reach areas. Experience the Stick With Us difference for yourself today!



SPRAY.



STICK.



GO.

General Information

- MEDICAL GRADE SILICONE ADHESIVE
- INCREASES THE ADHESION OF SKIN BARRIERS, CONDOM CATHETERS, SILICONE BASED PROSTHETICS AND TAPE BORDERED DRESSINGS
- HELPS TO PROVIDE A SECURE, WATERPROOF, SWEAT RESISTANT SEAL

Using Medical Adhesive Spray

1. Prepare your pouching system or desired item to adhere as instructed.
2. Remove any protective release liner or backing film from the adhesive skin barrier or item you wish to enhance the adhesive properties of.
3. Shake "Stick With Us" medical adhesive spray can well before using.
4. Spray directly on to the item you wish to adhere. Allow to dry for 5 minutes. Do not spray directly onto the skin.
5. Apply your product as instructed.



STOCK No. STK100

STOCK No. STK50

FAQ



Is it necessary to use your medical adhesive spray on the entire item I wish to adhere?

You can spray it on the entire surface of the skin barrier or item you wish to adhere. If there are particular areas that are not adhering well, you can use it only in those specific areas to increase its adhesive properties.



What is the best way to spray the adhesive?

Hold can so that the nozzle faces the product to be sprayed. Spray in short bursts until the surface of the product is coated evenly. 360° technology allows you to hold the can in any direction during use.



Can a second coat be applied?

Yes, if necessary, you can apply a second coat. Remember to wait five minutes for the second coat to dry before application.



Can I spray it directly on my skin?

This product should not be sprayed directly on skin. It is not necessary in order for it to be effective across all use cases.



Will the adhesive be difficult to remove from the skin?


"Stick With Us" medical adhesive should come off easily when it is time to be removed. If needed, a medical adhesive remover spray or universal remover wipe can help with ease of removal. For prosthetic devices, clean thoroughly and allow to air dry prior to next application.



TENACESMED.COM

stickwithus@tenacesmed.com

TEN ACED MED LLC IS PROUDLY IN COLLABORATION WITH

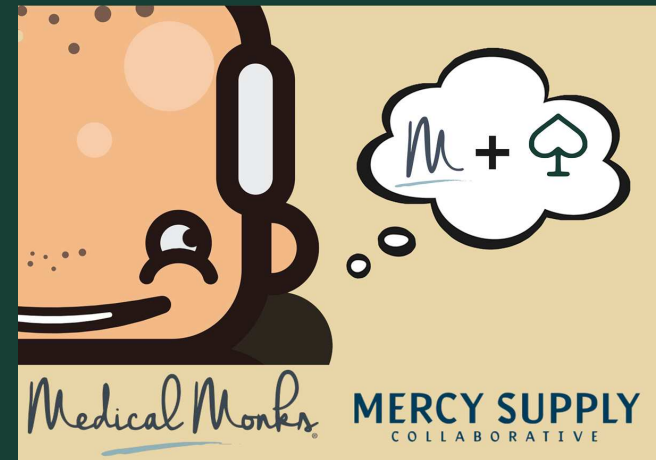

For Retail Orders:
MEDICALMONKS.COM
844-859-9400

For Wholesale Orders:
Mercy Supply Collaborative
mercyscb.com
844-859-9402

Routine follow-up with your healthcare professional is recommended. Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

The Ten Aces Med name is a trademarks of Ten Aces Med LLC. © 2024 Ten Aces Med LLC.