

Corsinel®

by Tytex

PREPARED FOR CUSTOMIZABLE HOLE

REIMBURSABLE UNDER
HCPC A4396

UPDATED BELT WITH PANEL



ABOUT BELT WITH PANEL ↘

- UNIQUE FRONT PANEL OFFERS OPTION FOR CUSTOMIZABLE HOLE FOR OSTOMY POUCH
- THE FRONT PANEL PROVIDES SUPERIOR HERNIA SUPPORT
- SUITABLE FOR BOTH 1- AND 2-PIECE SYSTEMS
- SEAMLESS TO ENSURE MAXIMUM COMFORT



4-WAY STRETCH



HIGHLY BREATHABLE



SEAMLESS KNITTING



FULLY OPENABLE

EUROPEAN-BASED PRODUCTION
FOR OPTIMAL QUALITY,
SUPPLY CHAIN EFFICIENCY
AND FLEXIBILITY



STANDARD
100
6234-7001
DTI

ARTICLE NUMBER: 3412

UPDATED BELT WITH PANEL

ABOUT THE BELT WITH PANEL

Corsinel Belt with Panel is designed to support and retain abdominal hernias. The belt offers maximum support to alleviate the uncomfortable pressure associated with a hernia, making daily and strenuous activities easier to perform. The uniquely designed panel at the front offers the opportunity to cut a hole to fit the individual stoma pouch. The 4-way stretch material of the belt will adapt perfectly to the body contour at all times. To mitigate the risk of allergic reactions, redness, or general skin irritation, biological safe materials have been tested for harmful substances according to OEKO-TEX STANDARD 100 is utilized. The belt is designed for all day use.

FEATURES

- Boat-shaped back to fit body anatomy
- Ready for hole customisation – template included
- Breathable material
- Maximum support for optimum hernia management
- 4-way stretch for optimum support and freedom of movement
- Unique panel with customized hole option
- Silicone Anti-slip pattern
- Hand pocket for easy application
- 100% seamless knitting with the purpose of avoiding skin discomfort for highest user comfort
- Tested for harmful substances according to OEKO-TEX STANDARD 100

INDICATION

For postsurgical use after construction of a colostomy, an ileostomy or an urostomy. For management of parastomal- or other abdominal hernias. For use with both 1- and 2-piece systems.

PRECAUTION

- Follow the Instructions for Use
- Follow recommendations given by your doctor or ostomy nurse
- Use the device according to the indications to avoid injury or damage
- Make sure the device is not compromising stoma and/or lung functions
- If your hernia is irreducible (cannot be returned by manipulation), seek the advice and guidance of your ostomy nurse before using the device
- If any unexpected discomfort, stoma, or skin irritation occurs, remove the device
- By urgent need for removal of the device, open with scissors
- Sizes overlap: If abdominal measurement lies between two sizes, choose the smaller size if you prefer extra tight-fitting garments, and the larger size if you prefer the garment less tight
- Ensure that the garment feels comfortable to wear, when sitting and standing



RECOMMENDED SIZES

SIZE	CM	IN
XS	74 - 82	29 - 32
S	78 - 90	31 - 35
M	86 - 98	34 - 39
L	94 - 106	37 - 42
XL	102 - 116	40 - 46
2XL	112 - 130	44 - 51
3XL	126 - 148	50 - 58

Measurement at the widest abdominal circumference



AVAILABLE COLOURS



COMPOSITION

76% PA
24% EL

AVAILABLE IN SIZES



CARE HANDLING



- Close hook-&-loop in the outer position before washing
- Wash with similar colours and in washing bag
- Wash at 104°F/40°C
- Washing or drying at temperatures higher than recommended will decrease durability and function
- Use of chlorine-based bleach or chlorine-based disinfectant will decrease durability and function