


SimPro™ Set

by Wellspect

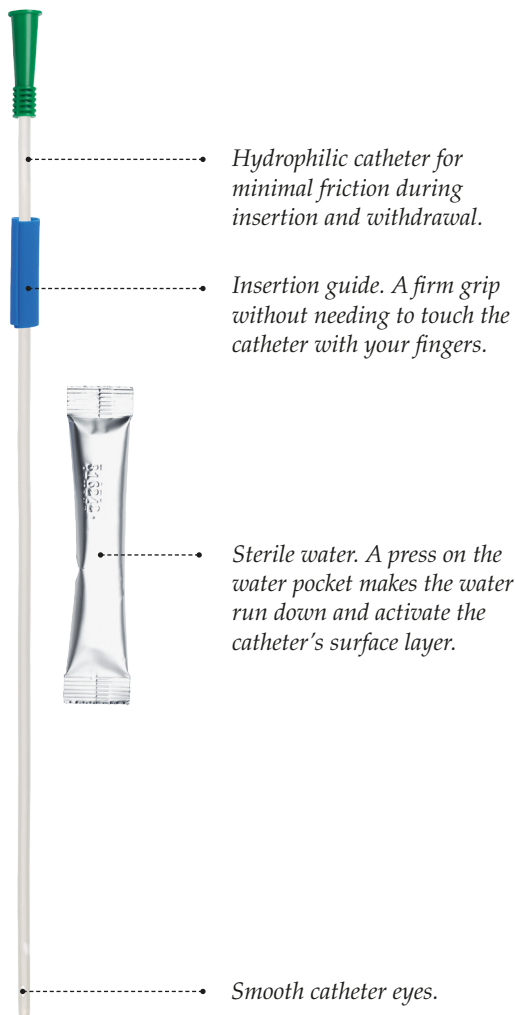


Hydrophilic catheter for minimal friction during insertion and withdrawal.

Insertion guide. A firm grip without needing to touch the catheter with your fingers.

Sterile water. A press on the water pocket makes the water run down and activate the catheter's surface layer.

Smooth catheter eyes.

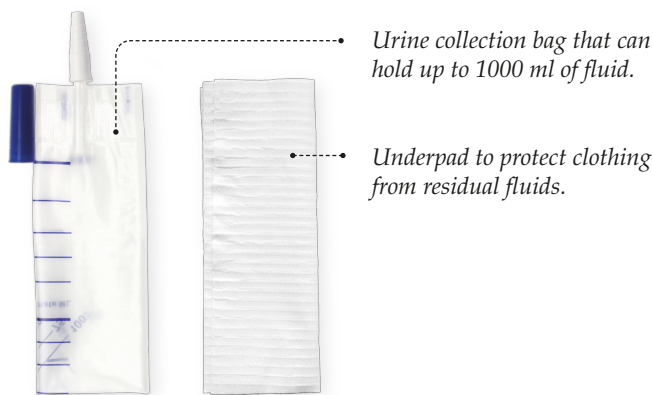

Urine collection bag that can hold up to 1000 ml of fluid.

Underpad to protect clothing from residual fluids.



Antiseptic cleaning pad to clean your urethral entrance.

Gloves to support clean catheterization technique.

Intended use

SimPro™ Set is intended for intermittent emptying of the urinary bladder. All SimPro™ hydrophilic urinary catheters are for single use only.

Sterilisation and shelf life

All catheters are sterilized using ethylene oxide. Information about the use-by date is printed on the package.

Package

10 catheters per carton and 4 cartons per shipping box.

Material composition

Hydrophilic surface coating: Polyvinyl pyrrolidone (PVP) and sodium chloride (NaCl).

Catheter: Polyvinyl chloride, medical PVC contains DEHP (Nelaton and Tiemann).

Water pocket: PET/Aluminum /LDPE and sterile water.

Item package: Polyethylene (PE) film, medical grade paper.

Gloves: not manufactured with natural rubber latex.

Product Range

Reference	Type	Length	Size
5331000	SimPro™ Set Female/Pediatric	8 in	10FR
5331200	SimPro™ Set Female	8 in	12FR
5331400	SimPro™ Set Female	8 in	14FR
5331600	SimPro™ Set Female	8 in	16FR
5301000	SimPro™ Set Male	16 in	10FR
5301200	SimPro™ Set Male	16 in	12FR
5301400	SimPro™ Set Male	16 in	14FR
5301600	SimPro™ Set Male	16 in	16FR
5351200	SimPro™ Set Coudé	16 in	12FR
5351400	SimPro™ Set Coudé	16 in	14FR
5351600	SimPro™ Set Coudé	16 in	16FR



Figure 1

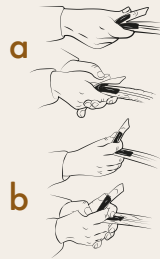


Figure 2



15+sec

Figure 3

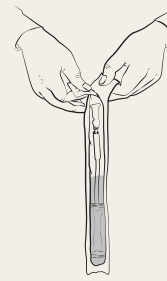


Figure 4



Figure 5

Instructions

Your doctor or nurse will explain how to use the insertion supplies; gloves, antiseptic pad and under pad, in order to perform sterile procedure.

1. Wash your hands before catheterization (Figure 1).
2. Open the package with care as it includes both the catheter, an attachable urine collection bag and the insertion supplies you need for a sterile procedure.
3. Remove the SimPro catheter from the package
4. Break the water sachet by using one of the illustrated methods (Figure 2):
 - a. by folding and squeezing (make sure not to fold over more than half the length of the water sachet to avoid water going the wrong way).
 - b. by placing your index finger on the water sachet, approximately half way down. Then use your thumb to fold the sachet around your index finger and squeeze.

Then let the water run down into the package.
5. Make sure that the catheter is soaked in water for at least 15 seconds, and get ready for catheterization (Figure 3).
6. Open the package by peeling the tabs on the connector side (Figure 4).
7. Attach the urine collection bag to the connector end of the catheter before removing the catheter from the catheter package. The catheter is now ready to insert.
8. (a) By using the blue handling strip to hold on to, you can perform catheterization easily without ever touching the catheter
(b) If needed, handling strip can be removed before or during catheterization (Figure 5).
9. When your bladder is empty, disconnect the catheter from the bag and discard the catheter. Hydrophilic catheters are intended for single use only. Reuse may cause urinary tract infections or urethral trauma.
10. Empty the bag either through the valve or use the cap provided to close the bag if you cannot empty it right away. (Please note that the cap may not totally seal the bag, so handle it with care.)