

INSTRUCTIONS

(cont.)

1. CONNECT

Simply connect the BobTail drain to your bag or pouch by pushing and twisting the connector (A) into the drain on the bag.



2. WEAR

Wear the BobTail drain all day if you choose (horizontally so that it easily comes out the zipper) or keep it in the provided bag until needed.



3. DRAIN

Simply pull out the Bobtail, remove the cap (D), aim, and open the valve (B) by pushing away from yourself. You can empty the bag standing or sitting, into a bottle, or toilet.



4. RETURN

When the urostomy bag is emptied, pull the valve (B) towards yourself to close, replace the cap (D), and return inside your clothing or to the provided bag.



5. CLEAN


Wash the BobTail drain daily with soap and water.

Dry thoroughly before using again.




6. REPLACE

The BobTail drain must be replaced every 14 days with a fresh device.

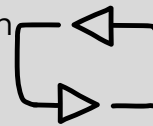

The BobTail®

from GV Solutions LLC

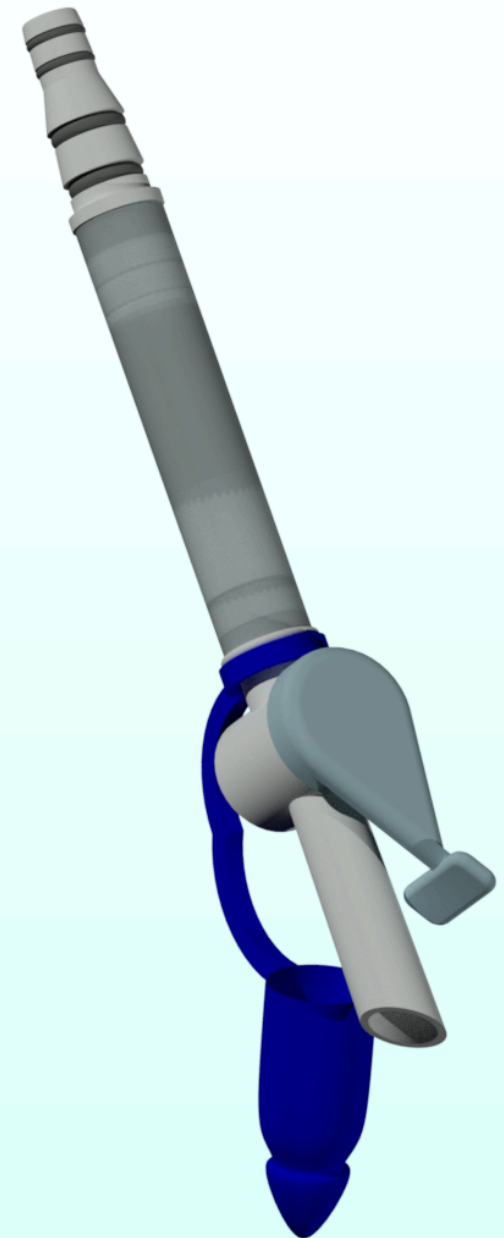

Video Instructions

(the-bobtail.com/instructions)



Contact Us

🌐 www.the-bobtail.com

☎ 717-319-5424

✉ info@the-bobtail.com

📍 P.O Box 127, Camp Hill, PA 17001

A Connector

Used to connect the BobTail to your Urostomy Bag. Insert inside of your bag's exit nozzle, twisting while pressing firmly until it is securely pressed within the exit nozzle. Give a tug to ensure that it is firmly seated. Twist while pulling on the connector to remove. Never pull from the Tubing, Valve, or Cap.

B Valve

Used to empty your bag. Turning the nozzle away from you (as shown) will empty the bag. Turning the nozzle towards yourself will stop the flow of fluid.

C Tubing

Flexible Medical Tubing enables transit of the fluid along with a comfortable fit.

D Cap

Used to cover the exit port to prevent making messes with leftover fluid on the end.

- GV Solutions LLC ensures the use of the BobTail drain for up to 14 days. Use after that may result in failure.
- When inserting the BobTail drain into the urostomy bag, pushing and twisting too hard may compromise the connection with too much pressure and voids the guarantee.
- The images used are representative of the final product.